

STERILE

Safe and efficient endodontics system



Bondent Group is a platform that provides integrated services for dental medical institutions.

We are committed both to improving world's level of dental medical technology and industry, and also to helping dentists succeed.

www.bondent.eu

 international@bondent.com

 **Europe:** +49 89 9545 5475
China: +86 187 5816 5369

 **Europe:** Bad-Berneck-Str.16
81549 München, Germany
China: 13th Floor, No. 655,
Fengzhou Road,
Jiading New Town,
Shanghai, China

 Bondent_Global
Bondent Europe
Bondent MEA
Bondent IN
bondent Turkey
 @bondent_global
@bondent_europe
@bondent_mea
@bondent_in
@bondent_turkey

Platinum V.EU

📌 A fine-tuned instrument for various clinical cases

An inspiring selection

Offers a range of files to effectively master canal anatomies from simple to complex. Start with an intuitive 3-file basic sequence, then get inspired by the wide range of files. Choose the ones to compose your own sequence and pursue your own therapeutic approach.

THE BASIC SEQUENCE Choose taper according to canal anatomy.

- If the canal anatomy requires further shaping, use larger sizes after reaching working length with 30.04 or 25.06.
- Choose Taper .04 for narrow and highly curved canals, including apical curvature.
- Choose Taper .06 for the majority of clinical cases.




Platinum V.EU: K-File	17/12vt	K-File	15/04vt	20/05vt	25/04vt	30/04vt	25/06vt
Speed :	N/A	300	N/A	350	350	350	350
Torque:	N/A	3	N/A	2.4	2.4	2.4	2.4

Safe and efficient endodontics system

Bondent presents its patent of cross section with flat design: the perfect compromise between cutting efficiency and debris removal

STERILE

Platinum V.EU

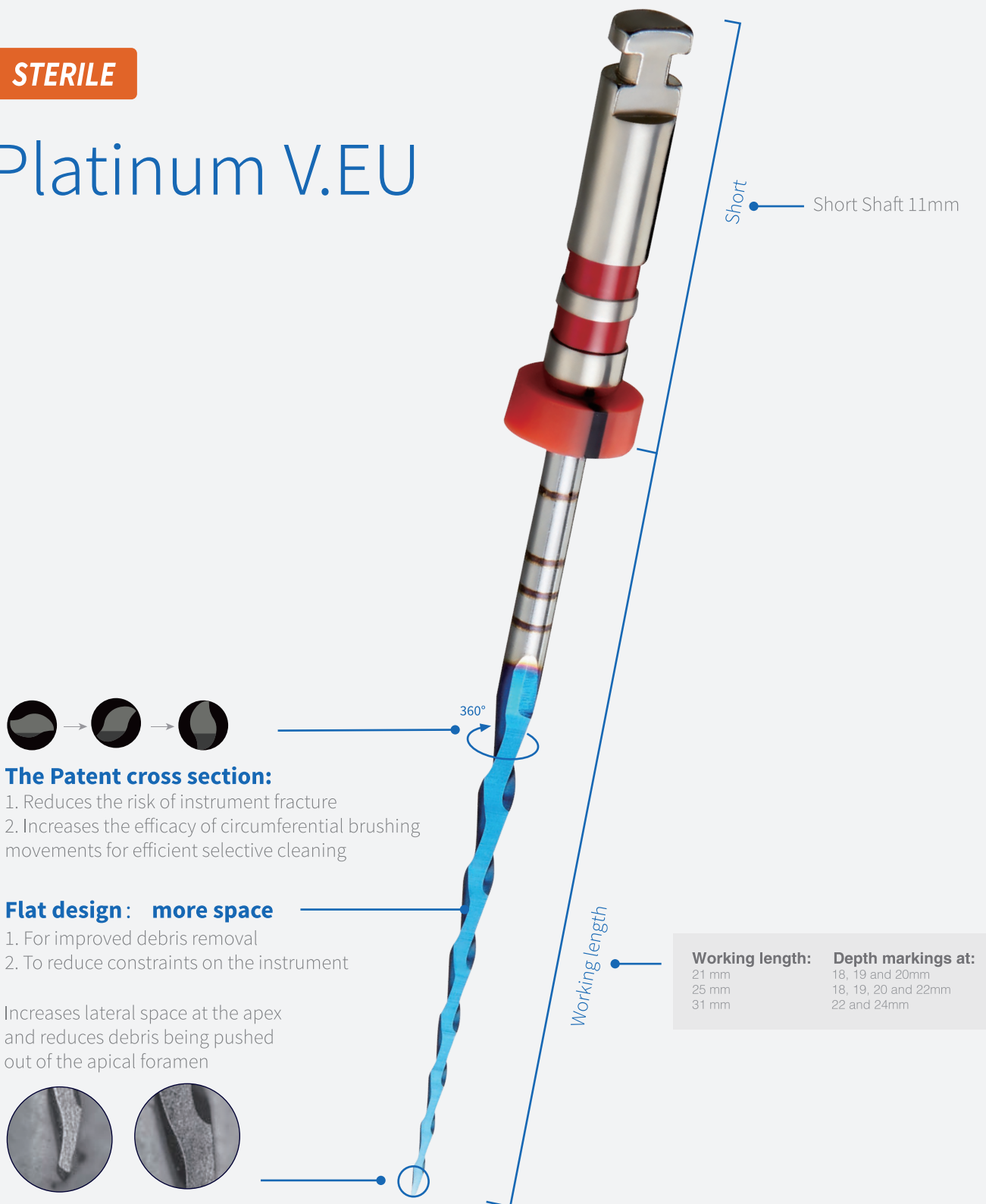

Sterile file clinic advantages

- 1 Sterile packing, single use no need for disinfection, each patient has his own files, reduce the risk of cross-contamination between patients
- 2 Sterile packing, single use no need for disinfection, save the nurse time, improve the efficiency

Sterile File Business Advantages

- The first company in China with the qualification to produce sterile files. The process uses purification workshop packaging and gamma ray sterilization to produce real sterile files.

Product reference

STERILE



17/12vt

19mm SPVEU012019017 ×6

STERILE

Assortment

21mm	SPVEU000021A01	×6
25mm	SPVEU000025A01	×6
31mm	SPVEU000031A01	×6

STERILE



15/04vt

21mm	SPVEU004021015	×6
25mm	SPVEU004025015	×6
31mm	SPVEU004031015	×6




iRoot Pro

SRCT2000	○
SRCT3000	●
SRCT2007	●
SRCT2008	●

Cordless Endo Motor
with integrated Apex Locator


STERILE

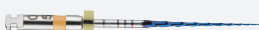

20/05vt

21mm	SPVEU005021020	×6
25mm	SPVEU005025020	×6
31mm	SPVEU005031020	×6

STERILE

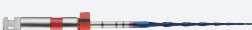

25/04vt


21mm	SPVEU004021025	×6
25mm	SPVEU004025025	×6
31mm	SPVEU004031025	×6



Standard Gutta Percha Points

20/04	UDGP004000015	×60
25/04	UDGP004000020	×60
25/06	UDGP004000025	×60
30/04	UDGP004000030	×60


STERILE

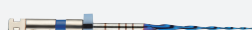

30/04vt

21mm	SPVEU004021030	×6
25mm	SPVEU004025030	×6
31mm	SPVEU004031030	×6

STERILE

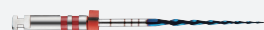

25/06vt

21mm	SPVEU006021025	×6
25mm	SPVEU006025025	×6
31mm	SPVEU006031025	×6